

# Rose K2

## Keratoconus Fitting Tips



	FLATTEN BASE	STEEPEN BASE	INCREASE DIAMETER	DECREASE DIAMETER	INCREASE EDGE LIFT	DECREASE EDGE LIFT	CONSIDER TORIC
LENS RIDING LOW	✓		✓		✓		
LENS RIDING HIGH		✓		✓		✓	
APICAL STAINING		✓					
3 & 9 O'CLOCK STAINING				✓			✓
SUPERIOR LIMBAL STAINING				✓			
FLUORESCEIN EDGE BAND TOO WIDE						✓	
FLUORESCEIN EDGE BAND TOO NARROW					✓		
GHOSTING OR CLOUDY VISION			✓				
POOR ACUITY	✓						
BUBBLING, ORANGE PEEL STIPPLING OR DIMPLE VEILING	✓			✓	✓		
POOLING AT CONE BASE	✓			✓	✓		
COMFORT					✓		
EARLY CONES			✓			✓	
ADVANCED CONES				✓	✓		

www.roseklens.com  
 email: rosek@menicon.com

